



# KiilTO PRO

# Replacing an empty bottle — Kiilto Pro Hygimon

Kiilto Pro Hygimon anonymously records the use of hand disinfectant dispensers and pump bottles and transfers the collected data to be evaluated.

Once in place, the sensor module attached to the dispenser or the pump bottle recognizes the transponder next to it automatically and records the use. When the bottle needs a refill, the swap for the new bottle is easy to do.

## PUMP BOTTLES

**(1) Remove the empty bottle and place a new bottle on top of the sensor module.** The disinfectant is now ready-to-use, no other actions needed.

## DIPSPENSERS (ON THE WALL)

**(1) Remove the empty bottle and insert the new one.** Once the new bottle is in place, **(2) hold the handle down for 5 seconds** so that the sensor module recognizes the new bottle. After this the disinfectant is ready-to-use.

